


Water-proof Alarm Button

ANW4

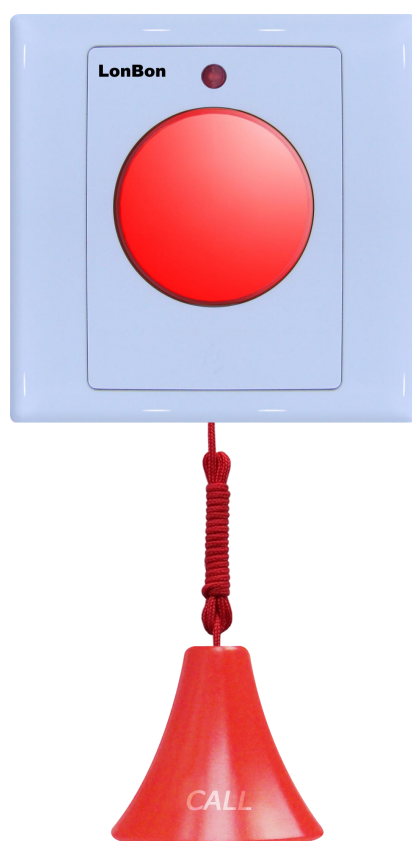

Features

The wall-mounted type is installed in the bedside or bathroom of the ward, for patients to call the Nurse in emergency situations.


Function

- Alarm mode: support pull rope and button alarm.
- Waterproof performance: IPX8 waterproof grade, supporting direct use under the shower.
- Alarm prompt: the ring indicator light will be on after the alarm.
- Connection mode: 3-core wire BUS connection

Technical Specifications

Working Voltage	12V-24V
Power	0.1W
Water Proof	IPX8
Alarm Type	NO (Normal Open)
Working Temperature	-20°C ~ +80°C
Working humidity	10% ~ 99%
Rope Length	0.6m
Dimension	86x86x32mm (LxWxH)
Packing Quantity	75pcs/Box

Dimension

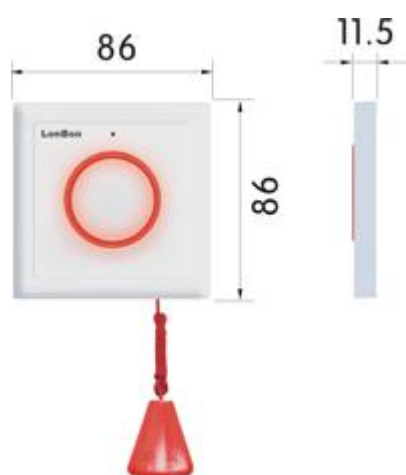


Position of assembly



How to Installation




Step 1. Take the lower cover and press the middle ring button with both hands, turn up the green edge of the cover, and take down the cover.

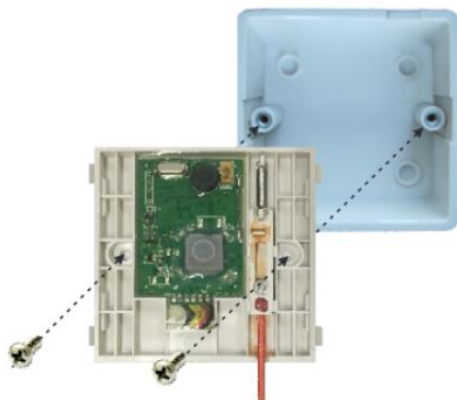

Step 2. Fix the bottom cover, screw the bottom cover of the button onto the embedded 86 box, tighten the screws properly, and then test the smoothness of pulling rope.

Note: over tightening of screws may cause deformation of cover and affect normal use.

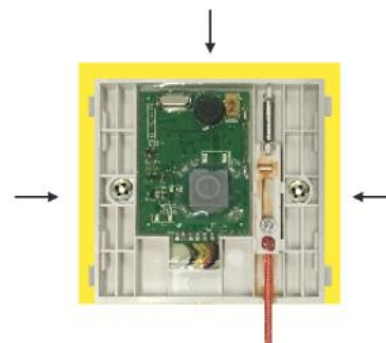

Step 3. Glue the upper, left and right sides of the bottom cover with water-proof glue to avoid water inside the equipment to accelerate the aging of the line; ensure that the long-term pulling of the cable causes the equipment to move, and finally fasten the cover.